

1



### Collect

Fill two cell-free DNA collection tubes between the minimum and maximum fill lines (**7-8.5 mls each**) with the provided materials.

2



### Order

Order the test at [vet.petdx.com](http://vet.petdx.com)\* and print out the submission form.  
\*If supported, complete order via your practice management software instead.

3



### Package

Label tubes with Pet Name & Requisition ID. Place them in provided tube sleeves and padding. Include submission form in biohazard bag.

4



### Ship

Use the OncoK9<sup>®</sup> box to ship the sample in a FedEx<sup>®</sup> Clinical Pak at **ambient temperature**. Postage is prepaid.



Immediately invert sample 8-10 times to ensure mixing



Do not refrigerate.  
Samples must remain at **ambient temperature**



Ship samples within 48 hours of collection